

pink or blue®


The Early Gender Test

NOTE: These instructions are for the Pregnancy Test component of this product, not the Gender Test. Do not open the Gender Test Kit until; 1) pregnancy is determined, 2) the mother is determined to be 6-weeks post conception (i.e., 8-weeks after the last day of the last menstrual period), and 3) you are ready to collect the finger-prick blood specimens for Gender Testing.

Pregnancy Test Instructions:

1. Remove the midstream pregnancy test from the foil pouch and familiarize yourself with the product.

2. Remove the Cap and place it over the Thumb Grip.

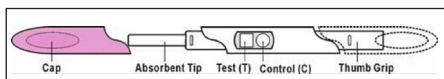

(The cap color may vary)

(See the illustration above)

3. Hold the midstream test by the capped Thumb Grip with the exposed Absorbent Tip pointing downward directly into your urine stream for at least 10 seconds until it is thoroughly wet. (See the illustration above)



NOTE: Do not urinate on the Test and Control windows. If you prefer, you can urinate into a clean and dry container, then dip only the Absorbent Tip of the midstream test into the urine for at least 10 seconds.

4. After removing the midstream test from your urine, immediately replace the Cap over the Absorbent Tip, lay the midstream test on a flat surface with the Test and Control windows facing upwards, and then begin timing.

5. As the test begins to work, you may notice a light red flow moving across the Test and Control windows. Wait at least 3 minutes for the red line(s) to appear. If no red line appears, wait one minute longer. Some positive results may be observed in 1 minute or less depending on the concentration of hCG. Do not read the result after 10 minutes.



Questions:

Call toll-free
1-866-6-WHAT-R-U
(1-866-694-2878)

Health Support Specialists available:

Weekdays 8:00 am–5:00 pm PST
Weekends 10:00 am – 4:00 pm PST
Or visit our website 24/7 for answers to common questions.

Consumer Genetics, Inc.,

562 E. Weddell Dr., Ste. 9,
Sunnyvale, CA 94089
www.consumergenetics.com
www.tellmepinkorblue.com
Toll Free: (866)-6-WHAT-R-U
Phone: (408) 734-2229
Fax: (408) 734-2221

INTERPRETATION OF RESULTS

